
COSMO-X

THE URBAN DREAM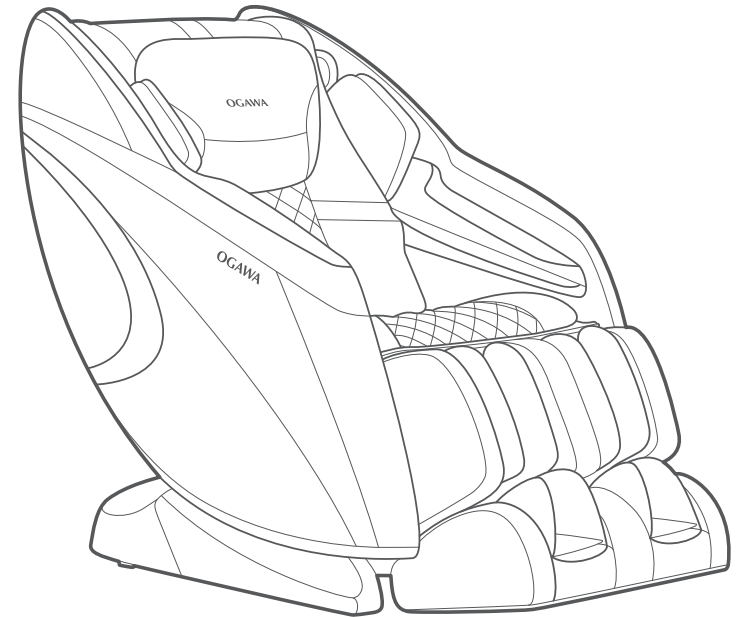

Ogawa is grateful
to have received
these awards:

Ogawa Wellness Hubs : Malaysia . Singapore . Vietnam . Philippines . Indonesia . Myanmar . India . China
Hong Kong . Taiwan . Turkey . Pakistan . Mauritius . Russia . France . Canada
Thailand . Korea . Australia . Dubai . K.S.A . Nigeria . Egypt . Lebanon . Denmark
Sweden . U.S.A

This user manual is also can be download via internet:www.ogawa.com.cn

User Manual
OG7266 使用手册

C O S M O - X

THE URBAN DREAM

User Manual
All information is correct at the time of print

Before Use

● Important safety instructions	01-04
● Grounding instructions	05
● Product parts	06-07
● Setup of the massage lounger	08-09
● Before use	10-12

How to Use

● Operation Guide	13-15
● After massage	16

Maintenance

● Cleaning and maintenance	17
----------------------------------	----

Q&A/Important notes

● Q&A	18
● Troubleshooting	19-20
● Specification	21
● Contact Us	23

Important safety instructions

Must be followed

These safety instructions must be strictly followed in use to avoid personal injuries and property damage are as follows:

■ All the safety instructions are described in compliance with degrees of the injuries or damage as a result of wrong use.

 Warning	Actions tending to cause serious injuries or death
 Caution	Actions tending to cause minor injuries or property damage

■ Pay attention to the symbols before safety instructions (an example is given below).

 Actions that must be absolutely forbidden	 Actions that must be done
--	--

 Warning	
 Must be followed	<ul style="list-style-type: none"> ● Before using the product, persons who are being treated or have any of the following symptoms should consult a physician. <ul style="list-style-type: none"> (1) Persons with a pacemaker or another in-vivo electronic medical device tending to suffer electronic jamming; (2) Patients with malignant tumors; (3) Patients with heart disease; (4) Persons proved to be unable to identify temperatures; (5) Pregnant persons or women who have just had childbirth; (6) Persons with perceptual disturbance because of diabetes or another serious peripheral circulation disturbance; (7) Osteoporosis patients, spinal fracture patients, or persons with acute painful diseases such as sprains or muscle strains; (8) Persons with injuries in the massaged parts; (9) Persons with a body temperature above 38 °C (with a fever) (For example, persons with intense acute inflammation symptoms such as listlessness, cold shivers and blood pressure change or persons in poor health); (10) Persons who are unwell or must have a rest; (11) Persons with an abnormal or bent vertebra; (12) Persons with protrusion of intervertebral disc; (13) Persons who feel very uncomfortable (except for the circumstances above) Otherwise accidents or physical discomfort may happen.
	<ul style="list-style-type: none"> ● If you feel physically abnormal while using or no effect after use, stop the product immediately and consult a physician about it. Otherwise accidents or physical discomfort may happen.
	<ul style="list-style-type: none"> ● Before being massaged or moving the backrest or the Lower Leg/Foot massage section, make sure there is no person or pet close to the product (in front, behind, below or on the side). ● Before use, make sure there is no foreign matter between the backrest and the back cover. Otherwise accidents or physical discomfort may happen.
	<ul style="list-style-type: none"> ● Before use, lift the back cushion and check whether fabric of traveling sections of the massage wheels or of other positions are damaged. (Even if the fabric is slightly damaged, stop the product immediately, pull out the power plug and send the product to repair.) If the product is used when the fabric is damaged, you may get injured or get an electric shock.
	<ul style="list-style-type: none"> ● In the first time of use, select AM Routine mode. ● Do not massage for more than 30 minutes each time. ● Do not massage a single part for more than 5 minutes. If any of these safety instructions is not followed, you may suffer an adverse effect and be injured.
	<ul style="list-style-type: none"> ● After use each time, turn the power switch to the “O” position ,and then pull out the power plug. Otherwise accidents or injuries may happen due to naughty behaviors of children.
	<ul style="list-style-type: none"> ● Follow the regulations about use of the socket and the wiring devices. ● Use a 220-240V~,50-60Hz power supply. (Do not use the product in a foreign country or connect it with a transformer without permission.) Otherwise electric shocks, product faults or fires because of overheat may happen.
	<ul style="list-style-type: none"> ● Insert the power plug thoroughly into the socket. Otherwise electric shocks or fires caused by overheat may happen.
	<ul style="list-style-type: none"> ● Remove such dirt as dust on the power plug at regular intervals. Otherwise fires caused by poor insulation due to factors such as moisture may happen. (Wipe the power plug with dry cloth.)

 Warning	
 Must be followed	<ul style="list-style-type: none"> ● In the case of any abnormality or fault, stop the product immediately and pull out the power plug to avoid smoke, fires and electric shocks. Abnormality and fault examples <ul style="list-style-type: none"> • The product fails to work after the power switch is turned to the ON position; • After the power line is moved, the product is discontinuously powered on; • In use, the product gives off a burning smell or an abnormal sound; • The product is deformed or abnormally heating; → In any of these abnormalities and faults, have the product inspected and repaired by an authorized service center immediately.
	<ul style="list-style-type: none"> ● Do not put the back cushion between the backrest and the back cover. Otherwise product faults may happen.
	<ul style="list-style-type: none"> ● When handling the product, hold its frame. Otherwise product faults or personal injuries may happen.
	<ul style="list-style-type: none"> ● The following persons should not use the product. Persons for whom massage is forbidden by doctors, such as persons with thrombosis, serious aneurysm, acute venous aneurysm, various skin inflammations (such as subcutaneous tissue inflammation) or various skin infections. Otherwise diseases may get worse.
	<ul style="list-style-type: none"> ● The following persons should not massage their waists. <ul style="list-style-type: none"> • Persons with acute lumbago, protrusion of intervertebral disc, or displacement of a lumbar vertebra; • Persons with deformation or other discomfort of the hip joint; Otherwise diseases may get worse.
	<ul style="list-style-type: none"> ● Do not put hands, arms or feet into the backrest's fabrics of traveling sections of the massage wheels. ● Children should not use the product. Keep them from playing on the product or climbing onto the Lower Leg/Foot massage section, the seat, the backrest or the armrests. ● Do not climb onto the backrest or the armrests. ● Do not lean or sit on the backrest. ● Keep your head from reachable areas of the backrest and the back cover. ● Persons unable to express themselves clearly or unable to operate the product should not use the product. ● When massing the neck, pay attention to moving directions of the massage wheels and do not massage with a too large force or massage front part of the neck. Otherwise accidents or personal injuries may happen.
	<ul style="list-style-type: none"> ● Do not damage the power line or the power plug. (Do not damage, machine, forcedly bend, twist or prolong the power line or the power plug, or place the power line or the power plug close to a heater, or tie or bind any heavy objects on the power line or the power plug.) A damaged power line or power plug may result in electric shocks, short circuit faults or fires. Please consult the shop from which you bought the product or a designated authorized service center about maintenance method of the power line or the power plug.
	<ul style="list-style-type: none"> ● When using the Warm massage wheels, avoid longtime contact with the skin in one position. ● Do not use the Knee Warm massage for a long time. Otherwise you may get low-heat burns. *Even if the temperature is relatively lower (40 °C~ 60 °C), low-heat burns may happen after longtime contact with skin (in spite of no obvious hot or painful feeling).
	<ul style="list-style-type: none"> ● Do not damage the Knee Warm (heater) section. <ul style="list-style-type: none"> • Do not insert pins or needles into the product. • Do not cut the product with a cutter. If the internal heater is damaged, fires or electric shocks may happen.
	<ul style="list-style-type: none"> ● Do not insert or pull out the power plug with wet hands. Otherwise electric shocks may happen.

 **Wet-hand operation forbidden**

 **Disassembly forbidden**

 Caution	
 Must be followed	<ul style="list-style-type: none"> ● The following persons (even if they are physically healthy at present) should consult a physician before using the product. <ol style="list-style-type: none"> (1) Persons with muscular atrophy due to aging or thin persons; (2) Persons feeling lumbago because of bone or viscus problems; (3) Persons tending to suffer sprains or bruising; (4) Persons with carsickness or seasickness or strong response when taking other transport means; (5) Persons who once received a heart operation or another visceral operation; Otherwise health may get worse. ● Before sitting on the product, make sure there is no foreign matter between any two parts of it. (Make sure there is no foreign matter on the back cover, the backrest, the seat surface, the Lower Leg/Foot massage section or inside surface of the back cushion.) Otherwise accidents, injuries, or product faults caused by clamping of hands, feet, or other articles by the product may happen. ● Do not use the product when wearing any hard decorative article on head (such as hair pin). ● If there is any hard article (such as mobile phone or key) in a pocket of the trousers, take it out before massage. ● Before hand or arm massage, remove watch, nail wrap, ring, etc. Otherwise injuries may happen. ● Before sitting on the seat, make sure the massage wheels are in the retracting positions. ● If you want to leave the seat during the massage, stop the product first. Otherwise accidents or injuries may happen. ● When pulling out the power plug, do not pull the power line but hold the power plug. Otherwise an electric shock or a short circuit fault may happen. ● Before moving the product, make the Lower Leg/Foot massage section back to the original position by adjusting its angle. Otherwise injuries may happen. ● When moving the product by pulleys, put articles such as mats on the floor. Otherwise marks may be left on the floor. ● Keep children from the retaining screws on the remote controller holder. Otherwise children may cause accidents or suffer injuries by eating it or playing with it. ● Before moving the product by pulleys, remove the obstructions close to it. Lift the seat surface of the Lower Leg/Foot massage section to a position a little lower than the waist and move the product slowly. Otherwise accidents or injuries may happen because of turnover of the product. ● In use, place the product horizontally. Otherwise accidents due to turnover of the product may happen. ● If any symptom such as skin inflammation, itching or swelling in use, stop the product immediately and consult a physician. Otherwise accidents or physical discomfort may happen. ● If the product is used again for massage after a period of out-of-service, read the Operating Instructions again and make sure the product can run normally first. Otherwise accidents or injuries may happen.
	<ul style="list-style-type: none"> ● Do not use the product and other devices at the same time. Otherwise accidents or physical discomfort may happen.
	<ul style="list-style-type: none"> ● Keep head, abdomen or skin from the massage wheels and do not put hands or feet between the massage wheels. ● Do not put the knees between the Lower Leg/Foot massage section. ● Do not fall asleep while using the product. Do not use the product after drinking. ● If air in the filling the air bag on the seat is not completely discharged, do not stand or jump on the seat or put things on it. Otherwise accidents or injuries may happen.

 Caution	
 Forbidden	<ul style="list-style-type: none"> ● Do not put hands or feet between the product and the Lower Leg/Foot massage section. ● Do not put hands, arms, or head into the seam of the Lower Leg/Foot massage section (footrest). For safety, admit the Lower Leg/Foot massage section after massage. ● Do not put hands, feet or head between the backrest and the back cover. ● Do not put pets on the product (such as its back cover, backrest and armrest). ● Do not pull out the power plug or turn OFF the power switch suddenly during massage. Otherwise injuries may happen. ● Do not stand or sit on the back cover. Otherwise personal injuries due to the product toppling may occur. ● Do not stand or sit on the armrests. Otherwise personal injuries may happen. ● Do not stand or sit on the Lower Leg/Foot massage section or the footrest. Otherwise accidents or injuries due to rotation of front of the footrest or turnover of the product may happen. If bottom of the Lower Leg/Foot massage section runs into the floor, the floor may be damaged. ● Do not drag or push the product after it is installed. Otherwise the floor may be damaged. ● Do not move the product while there is a person on it. Otherwise, accidents, or injuries may happen due to a toppling of the product. ● Do not use the product on heating devices such as electric blankets. Otherwise fires may happen. ● Do not put the remote controller on the product. Otherwise the product may be damaged.
	<ul style="list-style-type: none"> ● Do not use the product in a bathroom or another wet place. Otherwise electric shocks or product faults may happen.
	<ul style="list-style-type: none"> ● Do not sprinkle substances such as water on the product or its remote controller. Otherwise electric shocks, short circuit, or product faults may happen.
	<ul style="list-style-type: none"> ● Before cleaning or maintaining the product, pull out the power plug first. Otherwise electric shocks or injuries may happen. ● In the case of a power failure, pull out the power plug at once. Otherwise accidents or injuries may happen when power is restored. ● When the product is not used, pull out the power plug. Otherwise electric leakage fires may happen because of insulation aging caused by dust and moisture.
 Wet-hand contact forbidden	
 Product wetting forbidden	
 Pull out the power plug	

Setup of the massage lounger

Grounding instructions

- The product must be grounded to reduce the electric shock hazard because of functional faults of the product.
- The product has a power line with a grounding conductor and a grounding plug. The grounding plug must be inserted into an appropriate socket that has been correctly installed and grounded in full compliance with the local code and decree.


⚠ Caution

- If the grounding conductor is not correctly connected, electric shocks may happen. If you are not sure about correctness of the grounding, have it inspected by a professional electrician. Even if the grounding plug provided with the product does not match with the socket you have, do not refit it by yourself but ask a professional electrician to install an appropriate socket.
- The product has a grounding plug. Make sure the product is connected with a socket matching with shape of the grounding plug.
- Do not connect the product to a multi-plug.
- If the power plug is damage, it must be replaced by a special plug available from OGAWA or authorize agents.

1. This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
2. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision
3. The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance
4. The appliance is intended for indoor use only .

1 Before use (installation position)


- Make sure there is enough space for the tilting of the product.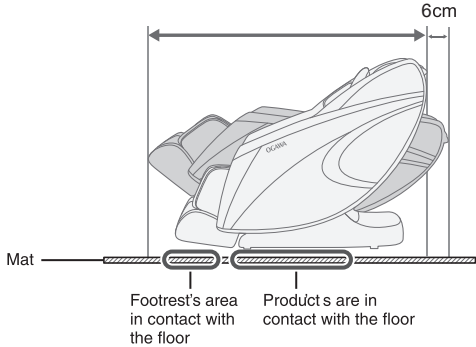

- Do not expose the product to direct sunlight or a high-temperature place (e.g., in front of heating equipment); otherwise its synthetic leather may fade or harden.
- Place a mat below the product to avoid floor damage. The mat should cover the product's area in contact with the floor and the footrest's area in contact with the floor (minimum dimensions: 200cm × 90cm). See the figure.
- Install the product close to a power socket.

2 Chair Moving Instruction

Method one: Hold the two side cover and lift up to moving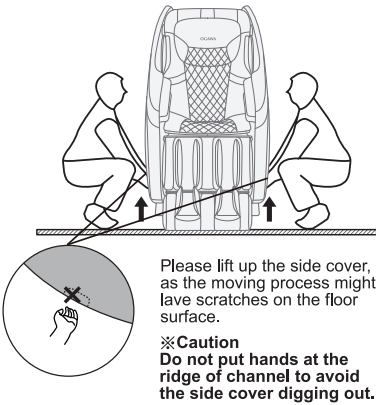

- The product is heavy and should be handled carefully by two persons at least to avoid back strain.

Product weight	Approx. 72.1 kg
----------------	-----------------

Put the product on the floor.

- Put down the product slowly and carefully to avoid foot injury.
- Do not release the footrest until the product is placed horizontally. Otherwise the footrest will rebound to its original position.
- It is suggested to pave an article such as a mat on the floor to keep it from damage.

⚠ Caution

- Do not drag or pull the product after it is installed.
- When handling the product, do not hold its frame. Otherwise product faults or personal injuries may happen.
- Do not move the product by pulleys directly on a floor. Otherwise marks may be left on the floor.
- Do not move the product while there is a person on it. Otherwise, accidents or injuries may happen due to falling down.
- Do not hold the foot massage section when moving the product. Otherwise injuries may happen due to sliding.

Method two: Moving by using slip bar underneath


⚠ Caution

Please put a mat under the chair while moving it, otherwise marks may left on the floor.

- 1 Press the ON/OFF button to return the back rest and footrest to its original position.
- 2 Slide the power switch to the "O" position and unplug the unit.

Tilt the massage chair downward and backward about 45 degrees before moving it, according to the methods shown


Note: The movement range should be moderate when pushing it down, so as not to cause damage to the hood due to the complete tilting of the chair.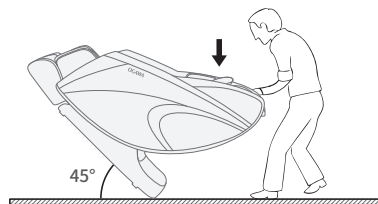

Before use

1 Check the surroundings

Make sure there is no people, pet and other object around the machine.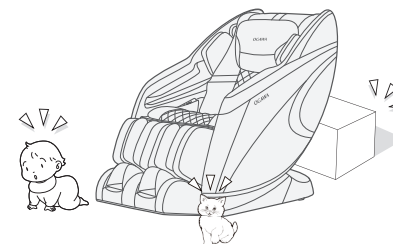


2 Make sure there is no object in the gap of the machine.

Before massage, please ensure that there is no object in the legs and foot and other positions.

⚠ Caution


- Do not stretch your hands, fingers, feet and head into the gap between the legs and the seat and between the legs and the armrest.
- Please ensure that there is no object between the parts of the machine before having a seat.
- Do not stretch your hands, arms or head into the gap between the backrest and the rear cover.
- Do not put your pets on massage chairs (seat, backrest, armrests, legs, etc.).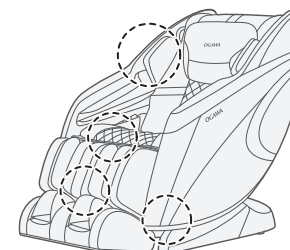


3 Power on.

Before massage, please ensure that there is no object in the legs and foot and other positions.

- 1 Insert the power plug into the power socket.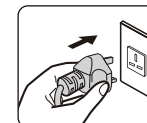

- 2 Set the power switch under the machine to the position "I" (open).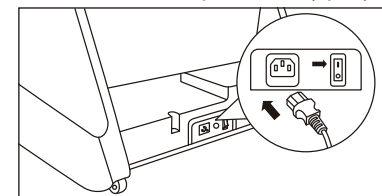

⚠ Warning

- Before use, please check whether the power cord and power plug are damaged or not.
- Please ensure that there is no object between the parts of the machine before having a seat. Please be sure to insert the power plug into the end. Otherwise, it may cause electric shock or fire due to overheating.
- Please use 220-240V ~ 50-60Hz alternating current. Do not use this product overseas or connect it to the transformer without permission. Otherwise, it may cause electric shock, machine fault or fire due to overheating.
- Do not plug and unplug power plug with wet hands.


Before use

4 Ensure that the massage wheel is in the storage position.

When the massage wheel is not in the storage position, press the “on / off” key to return it to the storage position.

Press the “on / off” key to return it to the storage position.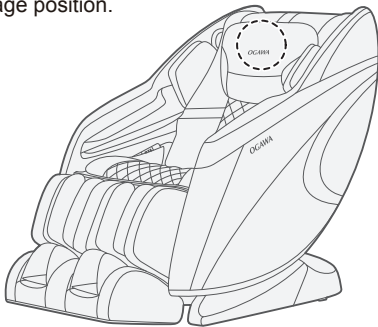


5 Ensure that the fabric is not damaged and cracked.

Key inspection area: Fabric of the backrest covered on the moving range of the massage wheel.


Caution

- Before use, you must first open the back cushion to check whether the fabric in the key areas is damaged. And the fabric in other parts should be also checked to make sure it is not damaged. (No matter how small the damage is, please stop using it immediately, unplug the power plug and send it to repair.) The use of the machine in case of damaged fabric may cause danger of injury or electric shock.


Key inspection area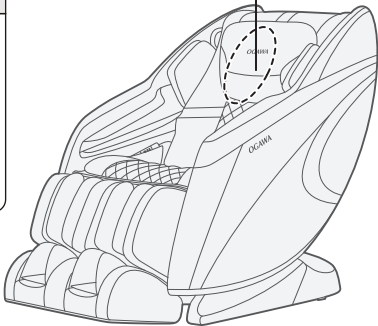

Product structure

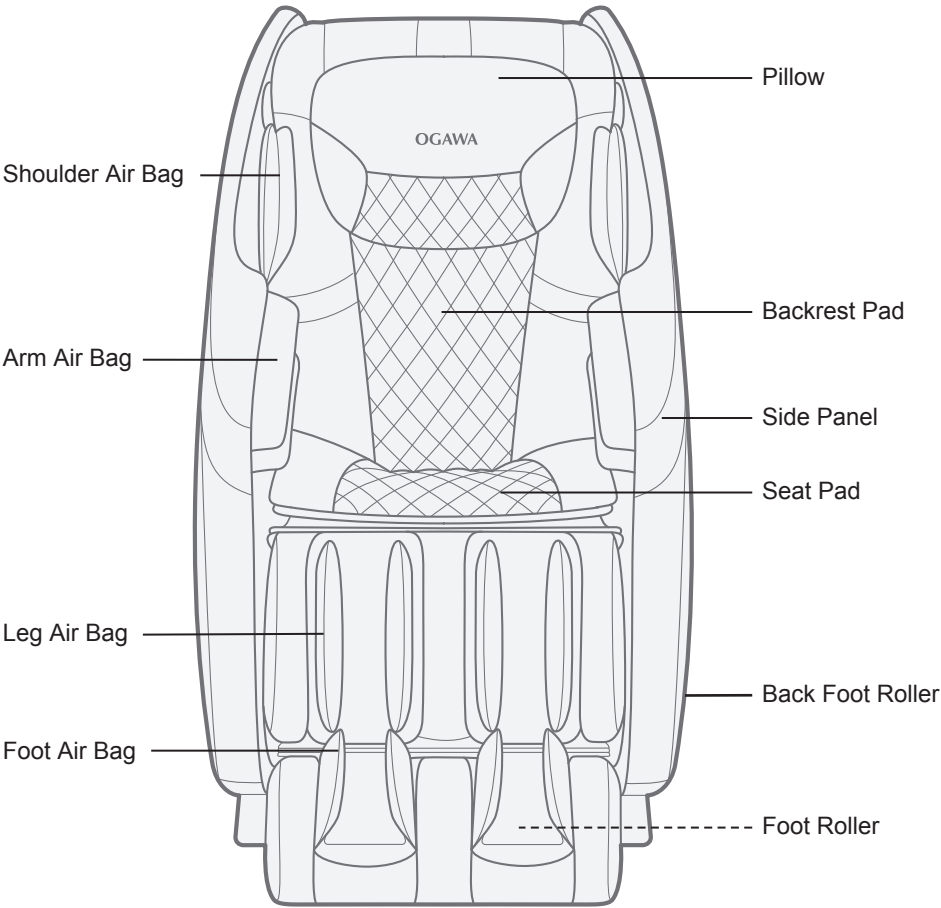Product structure

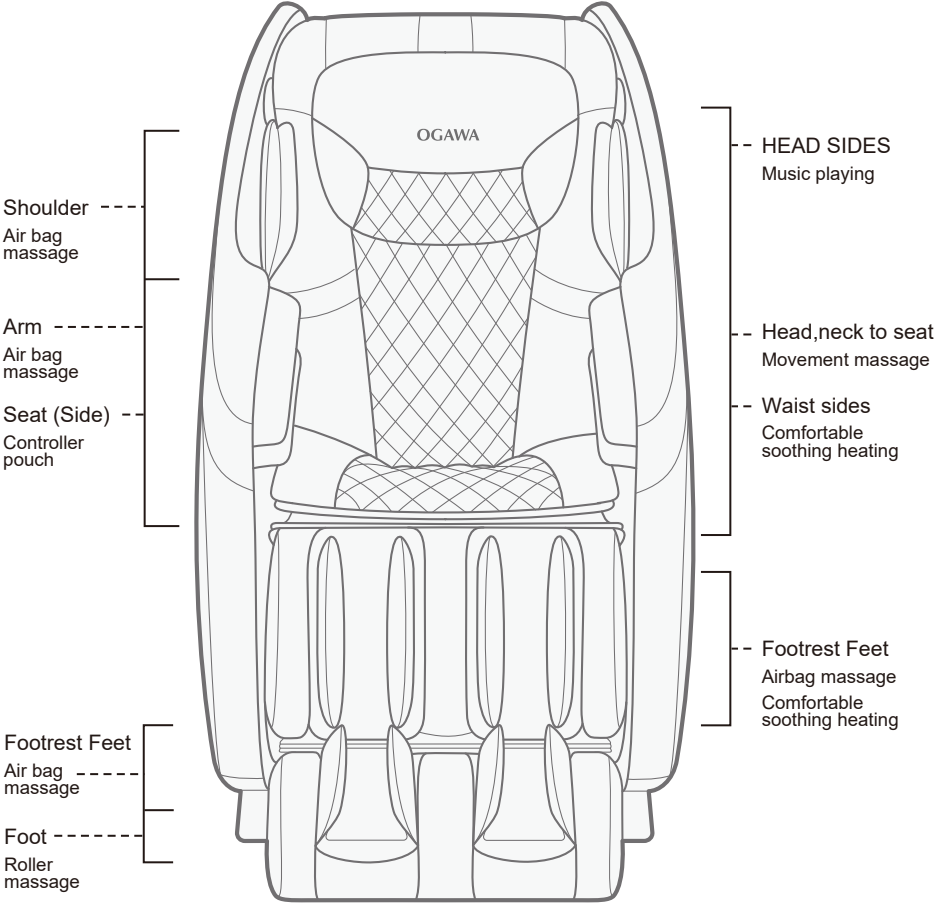Product structure

Operating instructions

I. Pay attention to the following items in the use of the product:

1. Power on and press the power switch under the seat frame of the massage chair to the position "I", and the massage chair is in reset state. The massage chair comes to the standby state. In the standby state, the red light of the On/Off control key keeps flashing, indicating that the power supply has been properly connected.
2. In the standby mode, only six functions, including start / stop, tilting forward / backward, legs lifting up/ down and zero gravity, are operable.

II. Massage chair control keys

1. Start / Stop

Whole machine massage function start / stop control key

When clicking the control key to start the machine, the red light and the screen of the hand controller will turn on. Then press any key to start enjoying the massage, and at the same time 15-minute timing function will be activated. When the product runs properly, click this key to turn it off, and the massage wheel, backrest and legs will be reset to the initial position.

Note: wobbling sleep mode lasts for 30 minutes, and the rest modes take 15 minutes.

2. Pause

Pause control key

Please press the pause key if you need to suspend the massage, at this time, all massage functions will be stopped. The orange lights of the pause control key will flash, and the timing function will be also paused. Press the control key again to continue the massage functions that have been paused, and the timing function will be also resumed.

3. Thermotherapy

Hip and Leg Thermotherapy Control

Press once to activate Thermotherapy at Hip area, Second Press to activate Thermotherapy at Leg Area (Hip area's thermotherapy will be deactivated), Third Press to deactivate Thermotherapy function.

4. Timer

Massage Timer Control

Auto Program timer at 15 mins per program

With every press, timer duration as such - 15mins > 20mins > 25 mins > 30 mins > 5mins > 10 mins > 15mins and so on.

5. Z-G

Zero gravity function start/stop control key

Press this key to start the mode of zero gravity 1, and press it again to change the mode to the zero gravity 2, and press it once again to stop the zero gravity function.

Note: zero gravity control key will be invalid when the Thai stretching function is activated.

6. Tilting forward / backward

Massage angle adjustment control key

The chair back angle can be adjusted through these two control keys. Press the tilting forward / backward key to adjust the angle of massage. When it is adjusted to a proper angle, press one of the two control keys to fix the position.

7. Legs lifting up/ down

Legs angle adjustment control key

The legs angle can be adjusted through these two control keys. Press the legs lifting up/ down key to adjust the angle of legs. When it is adjusted to a proper angle, press one of the two control keys to fix the position.

8. Automatic full-body massage function

When any of the six automatic massage programs is activated, the product automatically detects the back to achieve the optimal massage position. At the same time, you can manually adjust the massage position by pressing the up / down keys according to the needs.

6 automatic massage programs:

[Cosmo X] [Workaholic] [Glamour] [Text-neck] [Lumbar] [Thai Massage]

9. Spot/Partial

Spot/Partial Massage Control Button

Under manual massage program, press once for spot massage on shoulder area, press again for spot massage at back area; vertical 8cm massage area, third press to reactivate full body massage.

10. Upward/Downward

Position adjustment control keys for fixed-position and partial-position function

When the fixed-position/partial-position massage is activated, press and hold the Up/Down key to adjust the position of the massage wheel, and loosen the key for fixed-position massage when it reaches the position to be massaged.

Note: Start the automatic massage program, and if you need to adjust the shoulder height after shoulder detection, you can press the two control keys for fine adjustment within 10 seconds.

11. Massage Style Control

Back Massage style selection

Press once to activate full back massage function, with every press, massage style follows: "Rolling > Swedish > Tapping > Shiatsu > Patting > Tuina > Stop"

Mode Selection

Upon selection of massage style, you can select different modes. E.g When performing Rolling massage, pressing this button will change the mode to 'Rolling 2' mode, applies to other massage style.

12.Strength

Kneading massage intensity adjustment control key - 5-gear intensity adjustable

In the kneading massage mode, every time when this key is pressed,the massage intensity will be cyclically adjusted in the following orders: "Lv1>Lv2>Lv3>Lv4>Lv5"

13.Width

Tapping, Shiatsu, Clapping and Acupressure massage width control key -3-gear width adjustable

In the Tapping, Shiatsu, Clapping and Acupressure massage mode, every time when this control key is pressed, the massage roller width will be cyclically adjusted in the following orders: "Narrow>Medium>Wide" (6CM > 12CM > 18CM).

14.Speed

Swedish, Tapping, Shiatsu and Clapping massage function speed control key-5-gear speed adjustable

In the Swedish, Tapping, Shiatsu and Clapping massage mode, every time when this control key is pressed, the massage speed will be cyclically adjusted from slow to fast.

15. Seat Vibration

Seat Vibration Activation Control

Press once to activate seat vibration function, Press again to inactivate Seat Vibration Function

16. Airbag Massage

Airbag Massage Area Selection control

Press once to activate airbag massage, with every press the airbag > Arm > Shoulder > Ends. massage areas changes as such: Full Body > Leg.

17.Intensity

Air pressure massage intensity control key -5-gear intensityadjustable

In the automatic program or manualair pressure massage mode,every time when this control key is pressed, the air pressure massage intensity will be cyclically adjusted from weak to strong.

18.Foot roller

Sole rollerfunction start / stop control key

Press this key to start the sole roller massage function and the screen will display a sole roller icon. Press this key again to stop this function and the sole roller icon on the screen will go out.

19.Bluetooth connection

Name:MCxxxx xxxxx

Password:9999


The end of the massage

- 1 Retract the foot massage part to the original position.

For security, be sure to put your legs away after used.


Caution

- Children are forbidden to use this machine. In addition, children or pets are not allowed to play on the machine body, or climb up the legs, foot massage part, seat, backrest or armrest. Otherwise, it may cause accidents or injuries.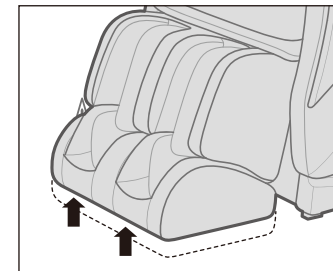


- 2 Method to retract the legs massage part:

- Ensure there is nothing in the legs massage part
- Slowly remove the legs from the legs massage part
- Press the back lifting up key on the controller to retract the legs massage part to the original position.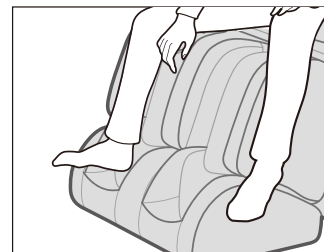


- 3 Put the controller in the storage bag on the right or left side of the machine seat.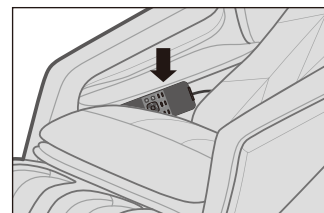


- 4 Cut off the power.

Warning

- Every time when you finish the use of it, slide the power switch to the position "O"(off), and unplug the power plug. Otherwise, it may cause accident or injury due to child's mischief.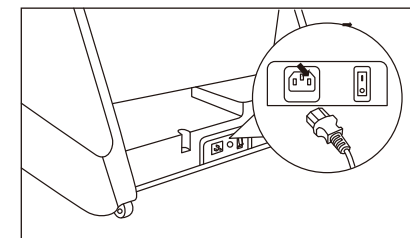

1. Turn "off" the power switch.
2. Unplug the power plug.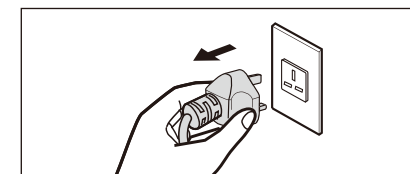


Cleaning and maintenance

Cleaning of the synthetic leather


Wipe the synthetic leather with a soft dry cloth.

- Do not clean the product with chemicals such as thinners, gasoline and alcohol.
- When using a leather maintenance product (cleaning cloth) sold on the market, follow its directions for use.
- If the synthetic leather is very dirty, wipe it with the method below.
 - ① Put a piece of soft cloth into water or 3% to 5% neutral detergent contained hot water and wring it out.
 - ② Wipe surface of the synthetic leather with the soft cloth
 - ③ Wipe the detergent with a piece of cloth once soaked in clean water and wrung out.
 - ④ Wipe with a piece of wrung soft cloth.
 - ⑤ Have the synthetic leather take natural air drying.
- If it is difficult to remove the stains on the synthetic leather, soak a piece of melamine foam bought from the market into a neutral detergent and use it to wipe the product.
- Do not dry the surface by a hair drier.
- Surface of the synthetic leather may be dried by fabric colors; therefore, be careful when wiping the product with denim or colored cloth.
- Keep these positions from long-time contact with plastics to avoid fading.
- Synthetic leather may discolor, so the position in contact with the hair dye used should be covered by a towel.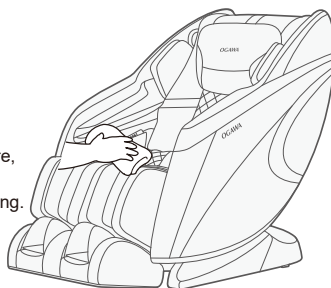

Cleaning of the plastic parts

- 1 Put a piece of soft cloth into water or neutral detergent contained hot water, wring it out and wipe the plastic parts.
 - Do not clean the product with chemicals such as thinners, gasoline and alcohol.
- 2 Wipe with a piece of cloth once soaked in clean water and wrung out.
 - Wring the cloth out before using it to wipe the remote controller.
- 3 Have the plastic parts take natural air drying.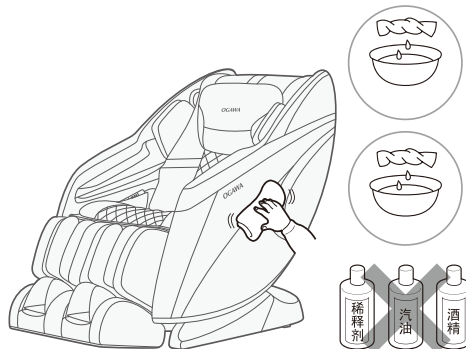

Questions and answers (Q&A)

Q Can one get sick often use the product?

A These persons should consult physicians before using the product.

Massage is the so-called "touch-pressure stimulation": Muscles are compressed to relieve their tension and promote hemokinesis. Some diseases may get worsened because of massage. Thus, consult the attending doctors before using the product.

Q How about the body height requirement for use of the product?

A The body height requirement differs with the specific body type. See the rough reference below when using the product.

- If a person shorter than 145 centimeters or taller than 185 centimeters uses the upper body massage function and the Lower Leg/Foot massage function of the product at the same time, it will be possible that his soles will have poor contact with the foot massage section or that his knees will need to be bent.

For a taller person with knee bending when using the product:

Select method **A** or **B** below:

- A** Tilt the backrest and lower the Lower Leg/Foot massage section
- B** Massage the upper body only with the help of the footrest. (Massage the feet and the upper body separately.)

For a short person difficult to touch the foot massage section:

Select method **A** or **B** below:

- A** Raise the backrest and lift the Lower/Foot massage section.
- B** Massage the upper body only with the help of the footrest. (Massage the feet and the upper body separately.)

Q Are there any things needing special attention during moving?


A Pay attention to the following things during moving.

- For convenience during moving, tilt backrest of the product till the lowest position.
- During handling, do not sit on the product.

① Press ON button power on the product.

② Press  to make the backrest completely tilt.

Press the button until backrest and footrest stop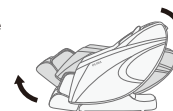


③ Turn off the power switch and pull out the power plug.

- If the original packing box is available, make sure the power line is not wound on the product, wind it and place it on the seat surface. When handling the product, keep the power line from falling off and rubbing with ground.
- If the original packing box is unavailable, properly keep other packing auxiliary materials and wrap the product with them to avoid scratch and vibration during handling.

Q Where is serial number of the product?

A It is on back of the main body.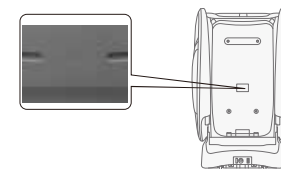

Q Mind the sounds of the product makes when operating.

A It is caused by the core structure.

Q Does it matter if the massage wheels have different massage intensities?

A This phenomenon possibly exists during massage because of the core structure.

To simulate the most natural massage effect, the product is designed with a core structure allowing alternate back massage by a left massage wheel and a right massage wheel. During massage, the massage wheels possibly have slightly different position on the back. That's why you have that feeling. It is not a fault.

Q The acupressure force is too large.

A Set the air bag intensity to be "1" through the Air Bag button. You can also lower the Lower leg/Foot massage section to reduce the acupressure force on the feet.

Q How about monthly power consumption of the product?

A Supposing you massage for 30 minutes in two times (15 minutes for each time) on each day, about 2.5 kilowatt-hours will be consumed each month.

Troubleshooting

After a fault happens, inspect and try the corresponding method below.

If the fault still exists after that, contact the shop from which you bought the product.

Fault	Cause and elimination method
The massage wheels stop during use (The product beeps and all icons and buttons on the remote controller flicker and then go out.)	<p>For safety, the massage wheels will possibly stop upon a too large external force during use.</p> <p>If all marks and buttons on the remote controller begin to flicker, it suggests an extreme abnormality. In this case, turn off the power switch and turn it on about 10 seconds later. If the massage wheels stop again, adjust your sitting posture and do not lean on the backrest completely. (Users, particularly those heavier than 100 kilograms, should follow relevant precautions.)</p>
The product cannot tilt, or the footrest can not be raised or lowered (The product beeps, the running light on the remote controller go out and the product stops)	<p>For safety, the product will stop upon an obstruction or a too large force.</p> <p>If all contents and buttons are flickering and the backrest or the footrest has stopped, turn off the power supply and turn it on about 10 seconds later and then operate the product.</p>
The massage wheels cannot reach the shoulders or the neck	<p>If the head is not in contact with the headrest, or the back is not in contact with the backrest, the shoulder position in body type detection will possibly be lower than the actual position. Sit in the deepest position of the product, lean your head on the headrest and operate from the beginning again.</p>
The left and right massage wheels have different heights (intensities)	<p>It is not a fault but a normal phenomenon because the product is provide with a massage core allowing alternate beats of the massage wheels.</p>
The product cannot run at all. ● Upper body (the neck to the waist) massage ● Lower leg/Foot massage	<ul style="list-style-type: none"> ● The power line is disconnected. ● The power switch is not turned on. ● No program is selected after ON button on the remote controller is pressed.
The product is damaged	
The power line or the power plug had abnormal heating	<p>To avoid accidents, contact an authorized service center.</p>


If above solution are not applicable, kindly refer to “Product Warranty Card” and contact OGAWA service center.

Troubleshooting (continued)

The sounds and feelings you mind in use of the product.

You will hear or have the following sounds or feelings when using the product. It is not abnormal phenomena but results of the product strcture.

- Sound when the stretching massage are retracted
- Sound and feeling when the massage wheels rub patterns on the fabric
- Sound of movements such as kneading and beat
- Friction sound of the running massage wheels
- Sound of up-down movement
- Sound of the conveyor belt
- Motor sound
- Sound of the squeezing massage wheels when they begin to stretch
- Sound when a person is sitting on the seat
- Friction sound between the armrests and the back cushion when the backrest is tilting
- Friction sound between the massage wheels and the fabric
- Sound of the inflator pump during Air Bag massage
- Sound of air exhaust during Air Bag massage
- Running sound of the solenoid valve at bottom of the product during Air Bag massage.
- Swing of the footrest
- Sound when the shoulders are stretched or moved
- Running sound of the air bags
- Sliding sound of the foot massage section
- Friction sound between the back cushion and the back cover leather when the backrest is tilting

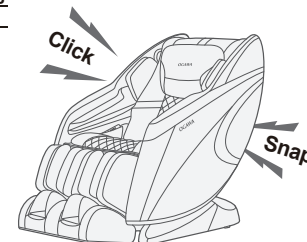If the sound is loud, it is also possible that the product has failed.

- In this case, have the product inspected and repaired by an authorized service center.

Warning

- **Do not refit, disassemble or repair the product without permission.** Otherwise fires, abnormal movements or personal injuries may happen.

Warning

- **In the case of any abnormality or fault, stop the product immediately and pull out the power plug to avoid fuming, fires and electric shocks.**
<Abnormality and fault examples>
 - The product fails to work after the power switch is turned to the ON position.
 - After the power line is moved, the product is discontinuously powered on;
 - In use, the product gives off a burning smell or an abnormal sound;
 - The product is deformed or abnormally heating
- In any of these abnormalities and faults, have the product inspected and repaired by an authorized service center immediately.

Specifications

Product Name	Massage Chair (COSMO-X)
Model No.	OG7266
Rated Voltage	220-240V~
Rated Frequency	50-60Hz
Rated Power Consumption	100W
Rate time	15 Minutes
Dimensions (L X W X H)	Upright: 1500X745X1120mm(L×W×H)
	Reclined: 1740X745X800mm(L×W×H)
Weight	Gross Weight: Approx. 87.1kg
	Net Weight: Approx. 72.1kg
Length of wire	Controller wire: 600 Centimeter
	Power supply wire: 180 Centimeter
Usage condition	Environment temperature:0°C~35°C
	Contrasting humidity:20%~80%
Storage Condition	Storage temperature:-5°C~35°C
	Storage humidity:20%~80%
Safety Feature	Equipped with overheated and power
Maximun body weight of user	Approx 100 kg

For a user heavier than 100 kilograms, the product may have bigger noise and its fabric housing may be worn quicker.

Contact Us

MALAYSIA

Healthy World Lifestyle Sdn Bhd
No. 22 Jalan Anggerik Mokara 31/47
Kota Kemuning, 40460 Shah Alam
Selangor Darul Ehsan, Malaysia
Tel : +603-5121 4286
Fax: +603-5121 4386

HONG KONG, CHINA

OGAWA Health Care International (HK) Ltd
31/F, YHC Tower, 1 Sheung Yuet Road
Kowloon Bay
Kowloon
Hong Kong
Tel: +852 3465 6262
Fax: +852 2410 0119

SINGAPORE

OGAWA Health-Care Pte Ltd
27 Tampins Industrial Avenue 5,
T5 @ Tampines
Singapore 528623

Tel: +65-6846 8588
Fax: +65-6846 7677

VIETNAM

OGAWA Vietnam Co Ltd
2nd floor, Petroland Tower,
12 Tan Trao, Tan Phu ward
District 7
Ho Chi Minh
Tel: +84 54133 222/54133223
Fax: +84 5416 1736

PHILIPPINES

O-Healthcare Solution Phil. Inc.
2303, 4th Floor, G&A Building,
Chino Roces Avenue Extension,
Makati City, Philippines
Tel : +632 889 0676 / + 632 844 6851
Fax: +632 812 0421

C O S M O - X

THE URBAN DREAM

User Manual
All information is correct at the time of print

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